**PURITAN
BENNETT**

**ADAM Nasal CPAP System
ADAM Nasal-CPAP-System
Sistema CPAP nasal ADAM
Système de ventilation CPAP nasal ADAM
Sistema per CPAP nasale ADAM
ADAM nasaal CPAP-systeem**

 Single Patient Use

Rx ONLY

REF

Nasal Ventilation Circuit Assembly	133283-00
Nasalatemungskreis, komplett	133283-00
Conjunto del circuito de ventilación nasal	133283-00
Ensemble du circuit de ventilation nasale	133283-00
Gruppo del circuito di ventilazione nasale	133283-00
Nasaal beademingscircuit	133283-00

ADAM Nasal CPAP System

Airway Delivery And Management

Single Patient Use

REF

Nasal Ventilation Circuit Assembly

133283-00

The ADAM Nasal Ventilation Circuit is designed to be used with portable ventilator's breathing circuit (Figure 1).

WARNINGS:

1. This circuit is intended for patient-assisted ventilation when treating hypoventilation. It is not intended for life-support ventilation.
2. Perform all the manufacturer's ventilator checkout procedures prior to connecting the patient to the ventilator.
3. Prior to applying the ADAM nasal ventilation circuit to the patient, adjust the portable ventilator to all prescribed settings.
4. Inspect before each use to insure that there are no leaks in the ADAM circuit, tubing or at any connections. Should leaks occur, discard and replace.
5. Use the 5 1/2" tube, supplied with the assembly, unless otherwise prescribed by a physician. Do not connect the exhalation valve near the airway outlet of the ventilator.

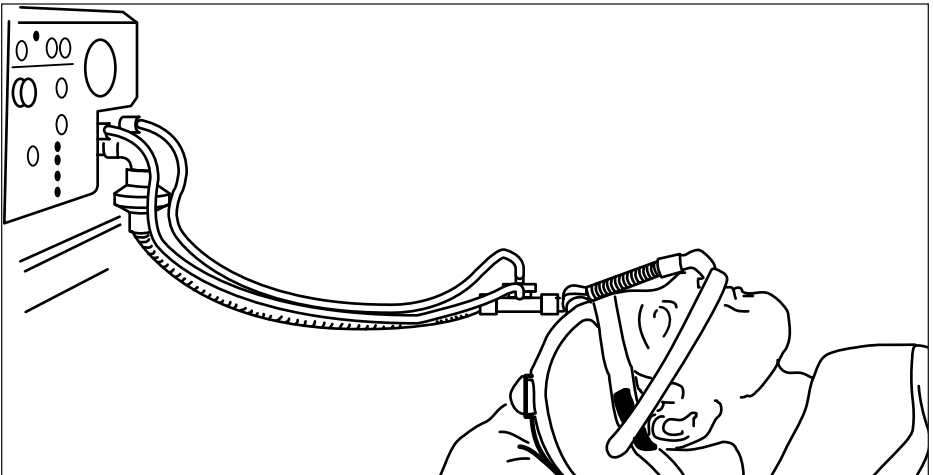
Figure 1

Nasal Ventilation Circuit Assembly

You will need the following components to assemble a complete nasal ventilation circuit (Figure 2):

- One ADAM nasal ventilation circuit assembly (P/N 133283-00)
- One ADAM headgear (P/N 133317-00)
- One pair of ADAM nasal pillows (P/N 616323-00, SM; P/N 616324-00, MD; P/N 616325-00, LG) or one pair of dilator pillows (P/N 616355-00, SM; P/N 616356-00, MD; P/N 616357-00, LG)
- One ventilator breathing circuit with exhalation valve (order with ventilator)

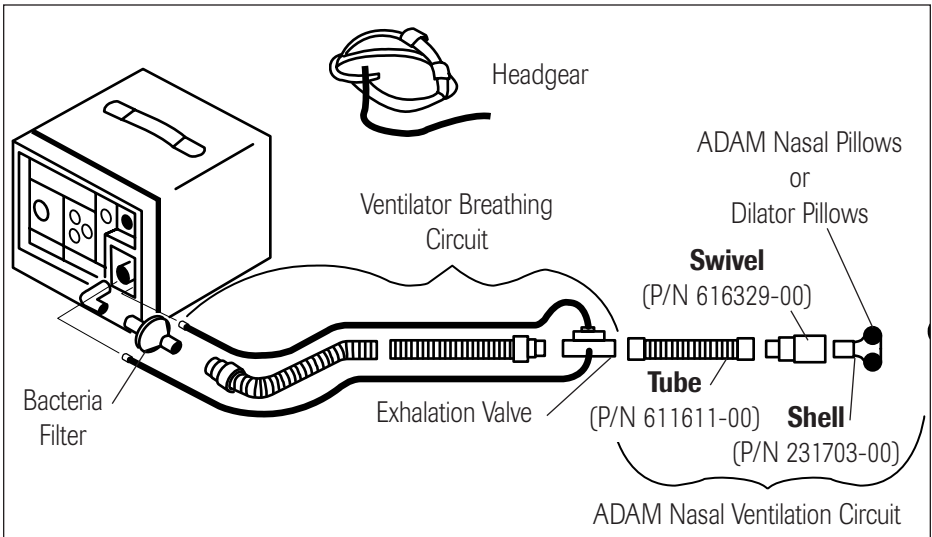
Figure 2

Headgear, Circuit and Nasal Pillows Assembly

1. Select the largest size pillow that seals and feels most comfortable.
2. Insert the pillows in the large holes located at one end of the ADAM shell circuit assembly. Be sure to insert only the first expanded ring in the plastic shell (Figure 3).
3. Fit the headgear to your head comfortably and then secure by adjusting and fastening the Velcro® straps in front (Figure 4).
4. Connect the ADAM nasal ventilation circuit to the exhalation valve on the ventilator's tubing circuit (Figure 2).

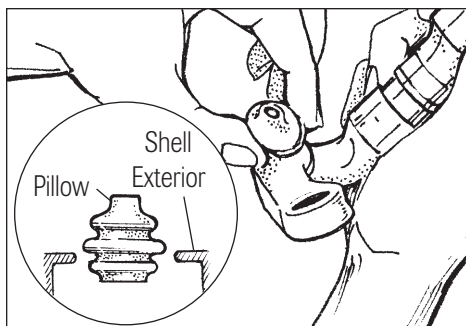
Figure 3

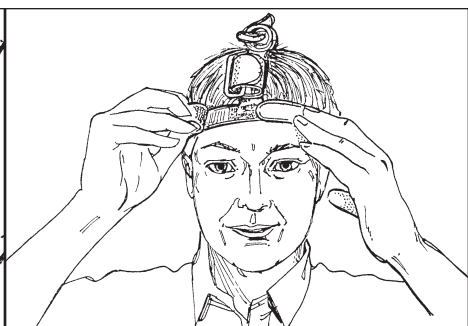
Figure 4

5. After fitting the headgear, remove and attach the circuit assembly to the headgear by wrapping the two short Velcro loops snugly around the tubing and fastening securely (Figure 5).

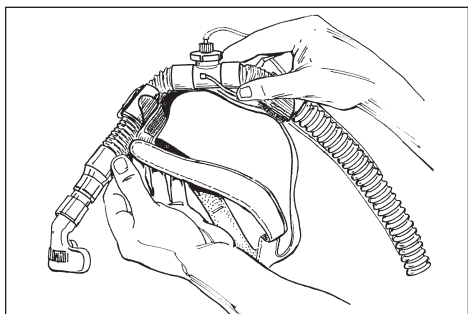
Figure 5

6. When you are ready for nasal ventilation, put on the headgear. Position the pillows in your nose, being certain the oval shape of the pillows is positioned for maximum fit (Figure 6). Readjust if necessary to allow proper positioning of the pillows in both nostrils. Secure the Velcro shell strap (Figure 7) and adjust the tension on each end of the strap to get a good seal in both nostrils. You do not have to disassemble the headgear to remove it from your head. Simply pull the shell and pillows gently away from the nose and remove the headgear like you would a hat.

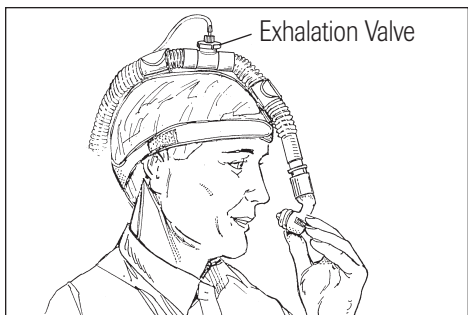
Figure 6

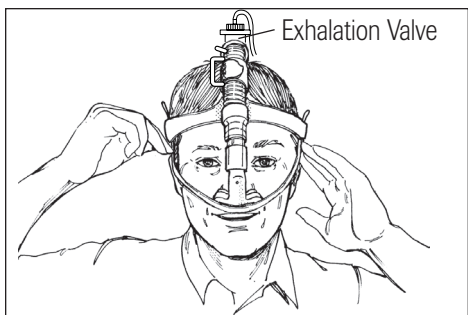
Figure 7

Cleaning Instructions

To increase the useful life of the ADAM circuit components, you should inspect all components daily and clean regularly.

WARNING:

Inspect and clean all components of the breathing circuit prior to use. Replace any breathing circuit components that show signs of damage such as cracks, holes or deformation.

Circuit Assembly

1. Remove headgear from the circuit assembly. Disassemble shell, pillows, swivel and 5 1/2" tube.
2. Wash shell, pillows, swivel and tube (both inside and out) in warm, soapy water (preferably with germicidal soap).
3. Rinse thoroughly and hang to air dry.
4. Follow the manufacturer's instructions for cleaning the ventilator's circuit.

CAUTION:

Do not allow water to contact or enter airflow device.

Headgear

1. Hand wash in mild detergent. Do not bleach.
2. Rinse thoroughly.
3. Hang to air dry.

Pillows

1. Clean weekly or more often as necessary using warm soapy water.
2. For disinfecting, use an aqueous activated dialdehyde-type disinfectant as recommended by your home care provider.

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©2003 Nellcor Puritan Bennett Inc. All rights reserved.
**Puritan-Bennett Corporation
Tyco Healthcare UK LTD