









# ComfortLite™ 2

## Helpful Tips for Successful Use

| ISSUE                                     | SOLUTION  |
|---|---|
| <p>Soreness or irritation in the nose</p> | <ul style="list-style-type: none"> <li>While the cushion is in the nose, squeeze the horizontal metal band on the bottom of the Pillows or Direct Seal™ Cushion to bring the pillows or bulbs closer together, or pull the metal band apart to separate the pillows or bulbs. This will redirect the air flow and reposition the cushion for a more comfortable seal and fit. (Figure 1)</li> </ul>  <p>fig. 1</p>   |
| <p>Leaking</p>                            | <ul style="list-style-type: none"> <li>Use angle (Figure 1) and vertical (Figure 2) adjusters to properly position the cushion.</li> <li>Try a larger size cushion.</li> <li>For the Pillows and Direct Seal Cushions, verify that the cushion is properly inserted onto the plastic ring of the exhalation chamber. If not (Figure 3), adjust cushion until properly inserted (Figure 4)</li> </ul>  <p>fig. 1</p>  <p>fig. 2</p>  <p>not properly inserted<br/>fig. 3</p>  <p>properly inserted<br/>fig. 4</p> |
| <p>Headgear sliding up on head</p>        | <p>Tightly adjusted upper straps will cause the back strap to rise up on the head.</p> <p>You can also try the following:</p> <ul style="list-style-type: none"> <li>Loosen the upper tabs of the headgear.</li> <li>Position back strap low on the head toward the back of the neck. (Figure 1)</li> <li>Refasten upper tabs.</li> <li>Move the headgear straps from the middle connection points to the top connection points. (Figure 2)</li> <li>For long or silky hair, wear the backstrap under the hair. (Figure 3)</li> </ul>  <p>fig. 1</p>  <p>fig. 2</p>  <p>fig. 3</p>                     |

**ISSUE****SOLUTION****Cleaning-Related Issues**

Cushion pulls away from plastic ring of exhalation chamber



Vertical Metal Band coming out of exhalation chamber

- Reinsert cushion onto the plastic ring. (Figure 1)



fig. 1

Slightly flex cushion and reinsert the metal band into the slot on the exhalation chamber. (Figure 1)

Do not squeeze the vertical metal band to bring the cushion closer to the nostrils (Figure 2). This can cause the vertical metal band to frequently dislodge from the exhalation chamber. Always be sure to use the angle adjuster (Figure 3) or the vertical adjuster (Figure 4) to position the cushion closer to the nostrils.



fig. 1



fig. 2



fig. 3



fig. 4

# RESPIRONICS®

Respironics Inc.: +1-724-387-4000

Respironics EAME: +33-1-47-52-30-00

Respironics Deutschland: +49-81-52-930-60

Respironics France: +33-2-51-89-60-00

Respironics Italy: +39-03-62-63-43-1

Respironics UK: +44-870-770-3434

Respironics Asia Pacific: +81-3-5280-9611

[www.comfortseries.respironics.com](http://www.comfortseries.respironics.com)

Respironics is a registered trademark and ComfortLite and Direct Seal are trademarks of Respironics, Inc. and its affiliates.

©2006 Respironics, Inc. and its affiliates. All rights reserved.

HOECH BD 5/17/06 MCI 4100655 PN 1035818