

# EasyFit & SomnoPlus Interface Sizing Template

Please Print Out at 100%! Do Not Scale!

**Nasal Mask Measurement Area**  
From bridge of nose to top of upper lip.

**Full Face Mask Measurement Area**  
From bridge of nose to bottom of lower lip.

**INSTRUCTIONS:**

**STEP 1:** Fold sizing template in half vertically.

**STEP 2:** Cut along dashed line to remove center (egg-shape) material and discard.

**STEP 3:** Align top of template with bridge of nose.

**STEP 4:** Select size based on appropriate measurement area.

**Size Selection:** XS, S, M, L, XL

**Nasal Mask**

**Full Face Mask**

Provided for your reference by  
**cpapXchange.com**



DeVilbiss Healthcare | 100 DeVilbiss Drive | Somerset, PA 15501 USA