

Less mask. **More you.**



Mirage™ FX. A transformation in nasal masks.

Make room for the only nasal mask you need. The perfect balance of lightweight comfort, premium performance and simplicity, the Mirage™ FX will transform your nasal mask business.

Instant patient appeal

With its compact frame, gentle cushion and ultra soft headgear, your patients will love the openness and comfort of the Mirage FX.

One mask fits all

One single cushion and frame size accommodates nearly all your patients. And, for patients who require a slightly wider fit, the Mirage FX is available in a Wide size. For your female patients who want a personalized option with a smaller headgear size, try the Mirage FX for Her.

Effortless and effective

Intuitive adjustment and contouring headgear make it unbelievably easy to achieve a perfect fit and secure seal on the first try.

Better business efficiencies


Reduce the time you spend on refitting and swapouts. Additionally, with fewer product codes and four simple components, inventory management is dramatically simplified.

Mirage™ FX
NASAL MASK

ResMed.com

Technology highlight

- New “butterfly wing” forehead support flexes to automatically adjust during fitting
- Streamlined design enables simple adjustment of seal on nasal bridge
- The soft, contoured headgear is all that touches the forehead—no silicone or plastic contact

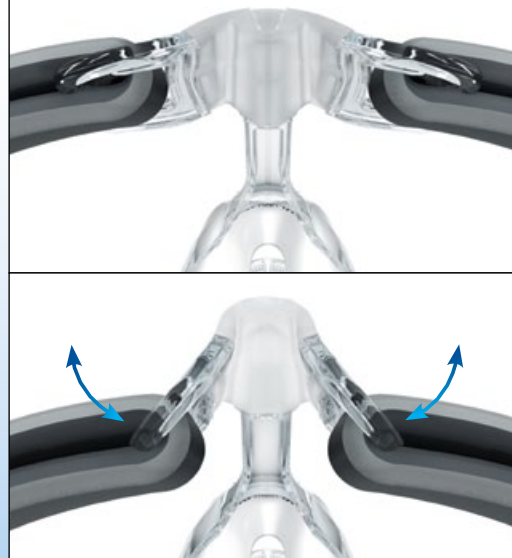

Flexible, “butterfly wing” forehead support
allows for a dynamic range of fit and adjustment; the plastic does not contact the skin

Lightweight, durable frame
is strong enough to stabilize the mask on the face, and soft and flexible enough to accommodate user movement during sleep

Diffused vent design
disperses air gently and quietly away from the bed partner

Squeeze-tab elbow
makes it easy to attach and detach the elbow from the frame

New 3D-contoured SoftEdge™ headgear
springs to life, ready to fit and wear

SoftEdge design
is kind to the face, reducing marks and discomfort

Dual-wall, Spring Air™ cushion
provides even pressure distribution with extra softness at the nasal bridge, reducing common discomfort in this sensitive area

Headgear loops
simplify attachment and detachment of headgear; retain optimal headgear settings



Components Guide

Give your patients a nasal mask that helps them accept therapy more easily and adopt it long-term. Make the **Mirage FX** the go-to choice for your nasal mask patients.

PRODUCT CODES

Complete system



62103 Mirage FX



62128 Mirage FX for Her



62118 Mirage FX Wide




Headgear

62110
62129 For Her

Cushion

62111
62125 Wide

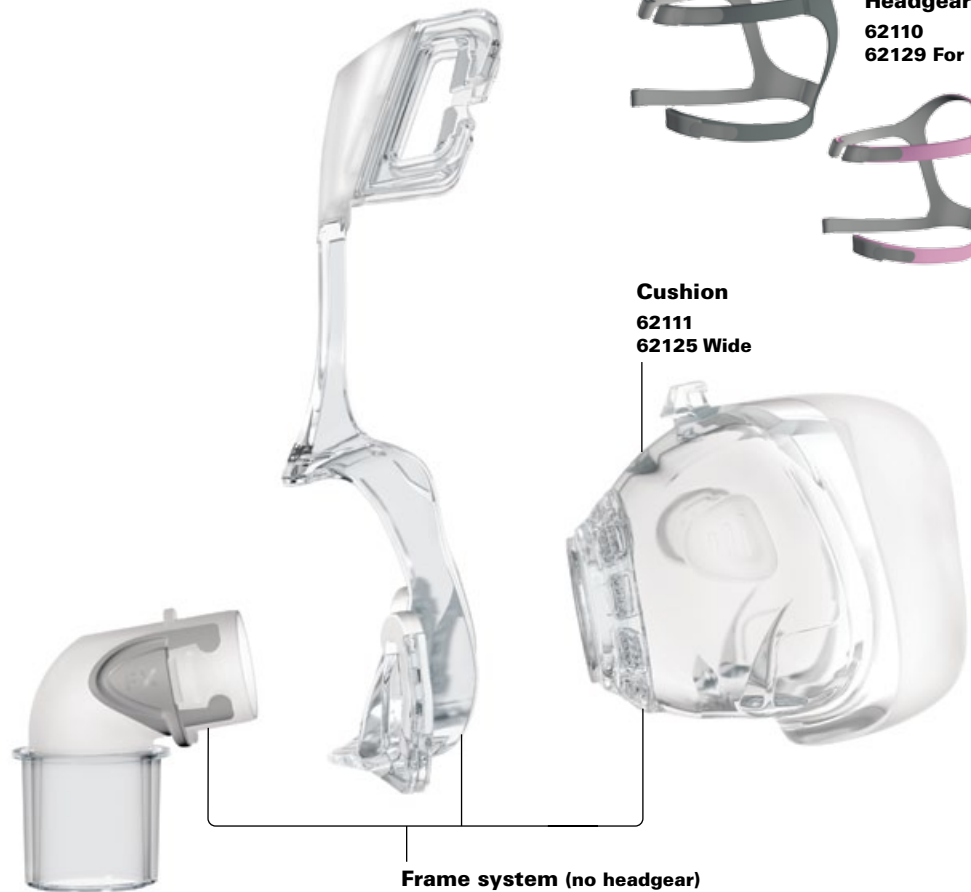

Frame system (no headgear)

62112 Mirage FX
62112 Mirage FX for Her
62126 Mirage FX Wide

REIMBURSEMENT CODES

HCPCS	Code Descriptor
A7034	Nasal mask used with positive airway pressure device, 1 per 3 months
A7032	Replacement cushion for nasal mask, 2 per month
A7035	Headgear used with positive airway pressure device, 1 per 6 months

 For details, refer to the **Mirage FX, Mirage FX Wide or Mirage FX for Her Nasal Mask User Guide.**

Mirage™ FX
NASAL MASK

Fitting Guide



- Prior to fitting the mask, remove the elbow from the mask by pressing the squeeze-tabs.



- Connect the elbow to the device air tubing.



- With both lower headgear straps released, hold the mask against the face and pull the headgear over the head.



- Bringing the lower straps below the ears, loop the headgear into the lower hooks on the mask frame.



- Unfasten the straps, pull the upper headgear straps evenly until they are comfortable and firm, and reattach the straps.



- Repeat step 5 with the lower headgear straps.



- Attach the combined elbow assembly and air tubing to the mask by pressing the squeeze-tabs and pushing into mask frame.
- Turn on the airflow device.

Pair the Mirage FX with ResMed's S9™ for an exceptionally quiet, comfortable system that fits your patients' lifestyles.



Mirage™ FX
NASAL MASK

ResMed.com