

Mirage Swift™ Nasal Pillows System

QUICK FITTING GUIDE



vent holes

1 Place headgear on head. Insert nasal pillows at the entrance of the nostrils.

Important: ensure vent holes are facing upwards



2 Adjust headgear straps; first the back and then the top.



3 Rotate pillow sleeve assembly forward or backward so the nasal pillows sit comfortably at the entrance of the nostrils.



4 Connect one end of the main air tubing to the main air generator and the other end to the swivel. Turn flow generator on.



5 The tubing may be positioned on either the left or right hand side. To change the tube position, swap the tube on one side with the cap on the other side.



6 You may choose to position the tube over the head (secured by the tube retainer if preferred).

Note: For complete fitting instructions, refer to the Mirage Swift Nasal Pillows System User's Guide.

Specifications may change without notice. 60861/1 04 04

MIRAGE
SWIFT™
NASAL PILLOWS SYSTEM