

## Mirage Swift™ II Nasal Pillows System

### Fitting Guide



Place the headgear on your head. Place the nasal pillows at the entrance of your nostrils.

**Important:** Ensure ResMed logo faces outward and away from your face



Gently tighten the back headgear straps. Do not overtighten.



Gently tighten the top headgear straps. Do not overtighten.



Adjust the angle of the pillows by rotating the sleeve assembly forward or backward so the nasal pillows sit comfortably at the entrance of your nostrils.



The short tube assembly may be positioned on either the left or the right side. To change the tube position, swap the tube on one side with the blue end cap on the other side.



Use the tube retainer (optional) if you prefer to have the short tube assembly secured to the headgear in the up position (over your head). The short tube assembly can be attached on either side of the mask, allowing for four different positions (left, right, up, down).