


 Sleepnet®
Fine, handcrafted sleep masks

Mojo™

**Full Face Mask
Size Gauge**

Small 50835
Medium 50836
Large 50837

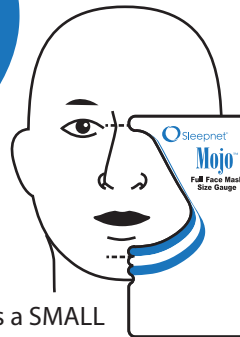


Example: Patient takes a SMALL

Hold size gauge up to patient's face and measure from indent on nose bridge to indent under lips. Relax jaw, with mouth slack and slightly open (about 1 1/2.5cm).

Provided for your reference by
cpapXchange.com

90102 Rev. B

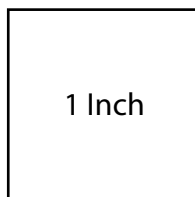

Print out template at 100%.
Make sure not to choose "shrink to fit" when printing.
To ensure your template is the correct size, measure the "1 Inch" square above.