

Provided for your reference by
cpapXchange.com

NEW UPDATES GIVE YOU SOMETHING TO ROLL WITH.

The easiest-to-use mask we've ever made is now an even easier choice to make. With updated features, our OptiLife nasal mask has more options and more flexibility than ever. So now, it's even simpler to fit more patients with a mask that's not only right for their lives... it's a great fit for yours.



OptiLife™
Nasal Mask

Envisioning tomorrow. Improving today.™

OptiLife™

Simple. Secure. Versatile.

Striving to provide you with even simpler, more sensible and more natural sleep therapy solutions, we've enhanced our highly popular OptiLife nasal mask to better fit the lives of patients, homecare providers and sleep lab professionals.

More than ever, patients will appreciate how easy OptiLife is to wear, clean and maintain. Updated mask tubing provides increased flexibility, assuring that OptiLife stays put, no matter what sleep position the wearer chooses. And now patients can choose the popular OptiLife nasal pillows design or an all-new CradleCushion™ style – an ideal choice for those who want a minimal-contact mask but who may not be comfortable with pillows cushions.

With its advanced headgear design and integrated chin support band, OptiLife can be put into place with one hand – almost as simply as putting on a ball cap. And adjustments can be made with little effort.

Sleep professionals will find that OptiLife makes it quick and easy to obtain proper fit and seal for a wide range of patients. Because OptiLife is packaged with multiple cushion sizes, first-time fit is straightforward, inventory management is enhanced, and callbacks are less likely.

The newly enhanced OptiLife has even more options and flexibility to fit your life.

Ordering Info

Item	Part No.	HCPCS
OptiLife Mask with Headgear and Pillows Cushions Sizes: P, S, M, L Pillows Cushions	1036800	A7034 A7035
OptiLife Mask with Headgear Sizes: P, S, M Pillows Cushions S, M Cradle Cushions	1036801	A7034 A7035 A7032
OptiLife Mask with Headgear Sizes: M, L Pillows Cushions M, L, LN CradleCushions	1036802	A7034, A7035, A7032
OptiLife Mask Only Sizes: P, S, M, L Pillows Cushions	1036804	A7034
OptiLife Mask Sub-Assembly with Headgear No Cushions	1036818	A7035*
OptiLife Mask Sub-Assembly No Cushions	1036819	*

*Requires addition of mask cushion for HCPCS – A7034

Replacement Parts	Part No.	HCPCS
Pillows Cushion, 1 per Pack, Size P	1036838	A7033
Pillows Cushion, 1 per Pack, Size S	1036839	A7033
Pillows Cushion, 1 per Pack, Size M	1036840	A7033
Pillows Cushion, 1 per Pack, Size L	1036841	A7033
CradleCushion, 1 per Pack, Size S	1036846	A7032
CradleCushion, 1 per Pack, Size M	1036847	A7032
CradleCushion, 1 per Pack, Size LN	1036848	A7032
CradleCushion, 1 per Pack, Size L	1036849	A7032
Headgear including Chin Support Band	1036850	A7035
Accessory Pouch	1036851	N/A
Tube Management Clip, 2 per Pack	1039967	N/A

Easy-to-adjust headgear design has no buckles – and it keeps the forehead and vision area open.

Pillows cushions are available in four sizes (petite, small, medium, large) to ensure an excellent seal.

Durable universal faceplate is easy to clean.

Exhalation port gives quiet and diffused airflow.





Two headgear slots give additional options to improve patient comfort and seal.

Chin support band provides a secure fit and ensures mask stability.

Simple attachment hub provides a secure cushion attachment and allows the cushion to rotate for an optimal fit.


Visual indicators provide orientation for proper cushion assembly.

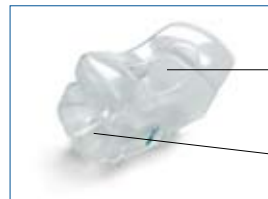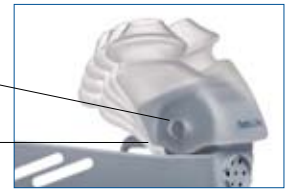

Generous landing area and deep arc of the CradleCushion cradle the nose for excellent comfort and seal. Available in four sizes (small, medium, large narrow, large).

Groove accommodates side-to-side movement.

Enhanced mask tubing moves freely with the wearer. The mask stays put for an optimum seal.

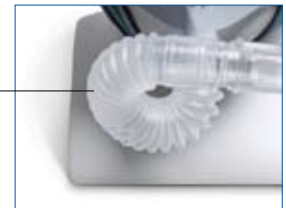

RESPIRONICS®

Envisioning tomorrow. Improving today.™

Customer Service: 800-345-6443 or 724-387-4000
 Respiromics Europe, Africa, Middle East: +33-1-47-52-30-00
 Respiromics Asia Pacific: +852-3194-2280
www.respiromics.com

CAUTION: U.S. federal law restricts these devices to sale by or on the order of a physician.

Respiromics is a registered trademark; CradleCushion, My Life and OptiLife are trademarks; and *Envisioning Tomorrow. Improving Today.* is a service mark of Respiromics, Inc. and its affiliates. Patents Pending.

©2008 Respiromics, Inc. and its affiliates. All rights reserved.

Hoech JJ 03/26/08 MCI 4101605 PN 1039633