***Phantom*[®] Nasal Mask
with User Instructions for 2 Point Headgear**

***Phantom*[®] Nasal Mask with User Instructions for 2 Point Headgear
Instrucciones de uso del casco de dos tiras con la máscara nasal *Phantom*[®]**

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Provided for your reference by
cpapXchange.com

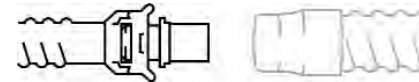
1

Mask tubing
 Tubo de máscara
 Maskenschlauch
 Tubulure du masque
 Slang masker
 Tubo della maschera

Mask plug
 Tapón de máscara
 Maskenstöpsel
 Prise du masque
 Maskerplug
 Tappo della maschera

5

Mask tubing
 Tubo de máscara
 Maskenschlauch
 Tubulure du masque
 Slang masker
 Tubo della maschera

CPAP tubing
 Tubo de CPAP
 CPAP-Schlauch
 Tubulure de l'appareil de
 pression positive expiratoire
 CPAP slang
 Dispositivo CPAP

2

6

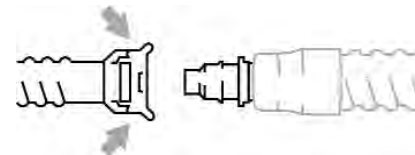
Nose clip
 Clip de nariz
 Nasenklemme
 Pince-nez
 Neusklem
 Clip naso

3

7

Squeeze
 Presionar
 Zusammendrücken
 Pincer
 Indrukken
 Premere

4

8

PHANTOM® Nasal Mask

Rx ONLY LATEX FREE MADE IN U.S.A.

GENERAL INFORMATION - WARNINGS AND CAUTIONS

- CAUTION:** U.S. Federal Law restricts this device for sale by or on the order of a physician.
- CAUTION:** Do not expose the PHANTOM® Nasal Mask to temperatures above 170°F (75°C). Do not sterilize.
- WARNING:** At low baseline pressure, the flow through the vent hole may be inadequate to clear all exhaled gas from the tubing. Some re-breathing may occur.
- WARNING:** Single patient use only. May be cleaned and used repeatedly on the same person but must not be used on multiple persons.
- WARNING:** This mask should be used with the positive airway pressure device recommended by the manufacturer, your physician, or a respiratory therapist. The mask should not be used unless the positive airway pressure device is turned on and operating properly. The mask vent hole should never be blocked. The vent hole allows a continuous flow of air out of the mask. When the positive airway pressure device is turned on and functioning properly, new air from the device flushes the exhaled air out through the mask vent hole. When the positive airway pressure device is not operating, exhaled air may be re-breathed. Re-breathing of exhaled air can, in some circumstances, lead to suffocation. This warning applies to most models of nasal positive airway pressure devices and masks.
- WARNING:** Do not use the PHANTOM® Nasal Mask with baseline pressure less than 3 cm H₂O or greater than 30 cm H₂O.
- WARNING:** Consult manufacturer for proper O₂ enrichment tubing.

CARE AND MAINTENANCE - GENERAL INFORMATION

The SleepNet Corporation PHANTOM® Nasal Mask is intended to be used with positive airway pressure devices, such as CPAP, operating at or above 3 cm H₂O for the treatment of adult obstructive sleep apnea.

CLEANING AND MAINTENANCE - SINGLE PATIENT USE

Cleaning is recommended on a daily basis. Wash the mask and components in warm water using a mild detergent. Rinse the mask thoroughly after washing. Visually verify that the vent hole is clear and allow the mask to air dry.

NOTE: Inspect mask prior to each use. Replace the mask if parts are damaged or gel is exposed due to tears or punctures.

NOTE: The gel cushion, although silky to touch, will not tear, leak or detach from the mask under normal use and above recommended cleaning and maintenance protocols. However, the gel cushion is not indestructible. It will tear, cut and rip if subjected to mishandling or abuse. Therefore, please handle your new PHANTOM® Nasal Mask with care.


The PHANTOM® Nasal Mask and its accessories are Latex Free.

User instructions for 2 Point Headgear

1. Attach tubing to mask, if not already assembled. The tubing and mask plug can be switched from right or left.
2. Thread the horizontal straps through the slots in the mask shell.
3. Place headgear on head with blue fabric facing outward. Pull straps outward to center the mask over nose. Affix Velcro tabs to straps and adjust as necessary.
4. Attach tubing to headgear with loop strap, if preferred.

NOTE: The mask should rest comfortably against the face and still provide an effective air seal. Pulling the straps too tight can cause skin irritation. The mask should not constrict your nose; you should be able to breathe freely and comfortably. If the mask and headgear are adjusted correctly and a problem persists, contact your healthcare professional.

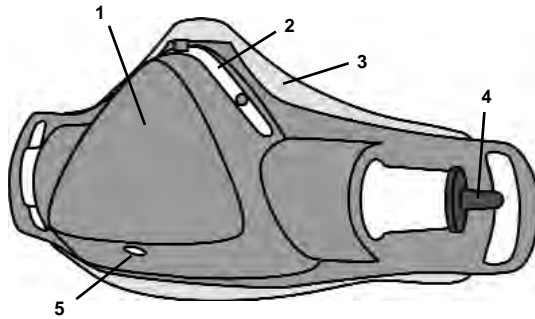5. Attach mask tubing to CPAP tubing; turn CPAP device on and verify airflow through mask.
6. Gently bend nose clip to create a comfortable fit and reduce leaks.

- | | |
|-----------------|-------------------------------|
| 1. Mask Shell | 1. Coque du masque |
| 2. Nose Clip | 2. Pince-nez |
| 3. Gel Cushion | 3. Coussinet rempli de gel |
| 4. Mask Plug | 4. Prise du masque |
| 5. Exhaust Hole | 5. Orifice d'évacuation d'air |
-
- | | |
|----------------------------|---------------|
| 1. Carcasa de la máscara | 1. Gelaatstuk |
| 2. Clip de nariz | 2. Neusklem |
| 3. Acolchado de gel | 3. Gelkussen |
| 4. Tapón de máscara | 4. Maskerplug |
| 5. Orificio de ventilación | 5. Uitlaatgat |
-
- | | |
|----------------------|--------------------------|
| 1. Maskenaußenhülle | 1. Guscio della maschera |
| 2. Nasenklemme | 2. Clip naso |
| 3. Gel-Kissen | 3. Cuscinetto in gel |
| 4. Maskenstöpsel | 4. Tappo della maschera |
| 5. Entlüftungslöcher | 5. Foro di sfiato |

REMOVING THE MASK AND HEADGEAR

7. Disconnect the mask tubing assembly from the CPAP tubing by pressing the sides of the quick connect. Part of the quick connect attachment will remain on the CPAP tubing.
8. Open one of the horizontal straps and detach from the mask.

 **PHANTOM**[®] is a trademark of SleepNet Corporation; Patent #6,019,10. CE Mark and identification number of Notified Body. Product conforms to the essential requirements of the Medical Devices Directive 93/42/EEC.

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This page intentionally left blank.

Sound pressure level of exhaust vent: 15 dB(A)
Resistance to air flow at 50 L/min: 1.4 cm H₂O; at 100 L/min: 3.4 cm H₂O

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This page intentionally left blank.

VENT HOLE PRESSURE/FLOW CHARACTERISTICS (Approx.)

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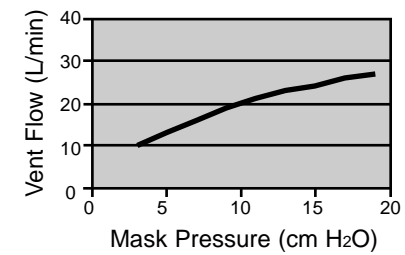
DECLARATION OF CONFORMITY

MANUFACTURER: SLEEPNET CORPORATION
5 MERRILL INDUSTRIAL DRIVE
HAMPTON, NH USA 03842

AUTHORIZED REPRESENTATIVE: MEDICAL PRODUCT SERVICES GmbH
BORNGASSE 20
35619 BRAUNFELS
GERMANY

PRODUCTS: PHANTOM NASAL MASK, IQ NASAL MASK,
MiniMe NASAL MASK & Mojo FULL FACE MASK

PRODUCT TYPE: POSITIVE PRESSURE BREATHING ATTACHMENT

PRODUCT OPTIONS: ACCESSORIES:
PHANTOM NASAL MASK: HEADGEAR

IQ NASAL MASK: HEADGEAR

MiniMe NASAL MASK: HEADGEAR

MOJO FULL FACE MASK: HEADGEAR

VARIANTS: N/A

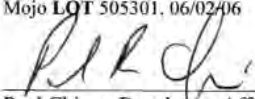
PRODUCT CLASSIFICATION: IIa, ANNEX IX RULE 2

WE HEREWITH DECLARE THAT THE ABOVE MENTIONED PRODUCTS MEET THE PROVISIONS OF THE COUNCIL DIRECTIVE 93/42/EEC FOR MEDICAL DEVICES (MDD 93/42/EEC).

STANDARDS APPLIED: MDD ESSENTIAL REQUIREMENTS

NOTIFIED BODY: TÜV PRODUCT SERVICE, GmbH

START OF CE MARKING: PHANTOM LOT 301658, 10/28/98
IQ LOT 301794, 12/27/99
MiniMe LOT 302046, 02/01/02
Mojo LOT 505301, 06/02/06

Paul Chiesa, Regulatory Affairs
Date: 12-8-06