

FITTING THE F&P PILAIRO IS EASY. FOLLOW THE SIMPLE SEQUENCE BELOW FOR BEST RESULTS:



### 1. Spread the Wings

Use your fingers to pull apart the pillows.



### 2. Position the Mask

Place the pillows on your nose until you feel a comfortable seal.



### 3. Locate Headgear

Pull on the auto-adjusting, one-size stretchwise headgear.



### 4. Test the Wings

Give the wings (pillows) a little wiggle for comfort.



### 5. Turn on CPAP device

The F&P Pilairo's AirPillow Seal will self-inflate and seemingly hover in and around your nose.