

## Swift™ LT for Her

NASAL PILLOWS SYSTEM

### Fitting Guide



1

- Holding the pillows at your nose, pull the headgear over your head.
- Headgear should sit midway between the ears and eyes.



2

- Adjust the top straps by pulling them evenly until comfortable.
- Do not overtighten.



3

- Repeat Step 2 with the back straps. The back straps can be positioned over or under the hair.
- Do not overtighten.



4

- Using the rotating barrel, position the nasal pillows at the most suitable angle.



5

- Connect the Swift LT for Her short tube to the device air tubing.
- Swift LT for Her is now fitted and ready for use.



#### Optional

- For alternate tubing positions, use the tube retainer.



- To attach, push it into the top headgear buckle.
- Clip in either the short tube or device air tubing, depending on flexibility needs.
- The tubing can be positioned to either side of your face or centered.
- If the pillows pull on your nose, adjust the tubing at the tube retainer.

 For details, refer to the *Swift LT for Her Nasal Pillows System User Guide*.

Manufacturer: ResMed Ltd 1 Elizabeth Macarthur Drive Bella Vista NSW 2153 Australia. Distributed by: ResMed Corp 14040 Danielson Street Poway CA 92064-6857 USA, ResMed (UK) Ltd 96 Milton Park Abingdon Oxfordshire OX14 4RY UK. See [www.resmed.com](http://www.resmed.com) for other ResMed locations worldwide. Protected by patents: AU 785376, DE 20321468.4, DE 20321469.2, DE 20321470.6, DE 20321471.4, DE 20321472.2, EP 1314445, NZ 562417, NZ 562418, NZ 562419, NZ 553756, NZ 553757, NZ 553822, NZ 553824, NZ 553825, US 6581594, US 6823865, US 7159587, US 7318437. Other patents pending. Protected by design registrations: AU 301943, CN 200430102469.3, JP 1243026, NZ 407161, US D552731. Other designs pending. Swift is a trademark of ResMed Ltd and Swift is registered in U.S. Patent and Trademark Office. © 2008 ResMed Ltd. Specifications may change without notice. 1013133/1 08 09