



Provided for your reference by  
**cpapXchange.com**



# Freedom to Sleep

## TAP<sup>®</sup> PAP Nasal Pillow Mask

Users like the TAP PAP Nasal Pillow Mask because:

- It is **very stable** regardless of the sleeping position
- Head straps are **optional**, no marks to the face or nose in the morning
- The nasal pillows fit **comfortably** without causing pressure sores on the nostrils
- TAP PAP is very **easy to breathe** on due to the design of the air pathway between the CPAP and your nose
- The mouthpiece makes TAP PAP the **easiest mask to fit** without needing frequent adjustments
- It is one of the **quietest** masks on the market at 25.7 dBA

[MYTAPPAP.COM](http://MYTAPPAP.COM)

Airway Management  
3418 Midcourt Road, Suite 114  
Carrollton, TX 75006, USA

866.264.7667

[MYTAPPAP.COM](http://MYTAPPAP.COM)

To visit our website  
with your smartphone,  
open your QR Code  
Reader App and scan:



WITH THE

## TAP<sup>®</sup> PAP Nasal Pillow Mask

**AM**  
AIRWAY  
MANAGEMENT

**AM**  
AIRWAY  
MANAGEMENT

## TAP<sup>®</sup> PAP Nasal Pillow Mask

More freedom. More comfort. More sleep.

### Experience Better Sleep

The TAP PAP Nasal Pillow Mask is designed to give you the *Freedom to Sleep*. Unlike most CPAP masks TAP PAP does not require headgear.

TAP PAP Nasal Pillow Mask is held securely in place by your upper teeth with a special ThermAcryl™ mouthpiece. The mouthpiece is easy to fit by warming in hot water and placing on your upper teeth.

Once you have fit the mouthpiece, the mask swings into place and you simply tighten the nut.

This means you do not have to adjust headgear and the mask during the night or on a nightly basis.

The end result is a comfortable mask fit that is securely held in place throughout the night. The nasal pillows do not need to be tight to your nose due to the overall stability of the mask.

*Sweet Dreams!*

### Get Serious About Sleep

Using your CPAP every night requires an interface that delivers comfort and performance. The TAP PAP Nasal Pillow Mask gives you those qualities and much more.

- TAP PAP is very stable without headgear
- TAP PAP will not leak
- TAP PAP is one of the quietest masks (25.7dBA)
- TAP PAP does not require nightly adjustments
- TAP PAP is backed by a 30-day satisfaction guarantee

### Get to Know the Key Components



**AM AIRWAY**  
MANAGEMENT

[MYTAPPAP.COM](http://MYTAPPAP.COM)