


Ultra Mirage II Nasal Mask, Mirage Activa Nasal Mask Fitting Template

How to use the fitting template:

1. Cut out template
2. Cut out nasal holes
(hint: folding each nasal hole in half makes it easier to cut)
3. Follow directions on template to determine appropriate cushion size

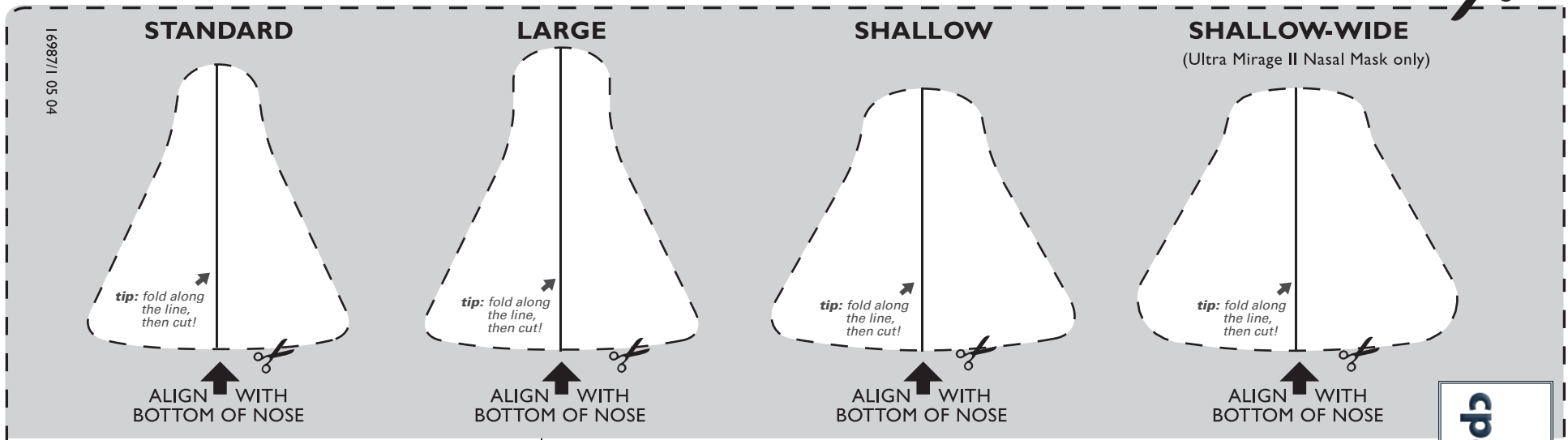

Ultra Mirage II Nasal Mask,
Mirage Activa Nasal Mask

FITTING TEMPLATE


RESMED

Note: Mask fitting templates should be used to aid patients in the fitting of CPAP masks. Altering the size or dimensions of the original online PDF in any way may compromise the quality and accuracy of this template. The use of this template is not intended to replace the advice of a medical professional, sleep technician or HME dealer.

Follow these two steps to choose the correct size:




1 Check the nostril width for clearance

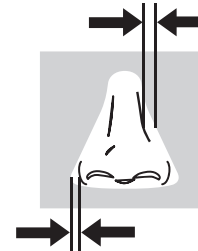

2 Check the nasal bridge width for clearance.

cpapXchange.com

Provided for your reference by

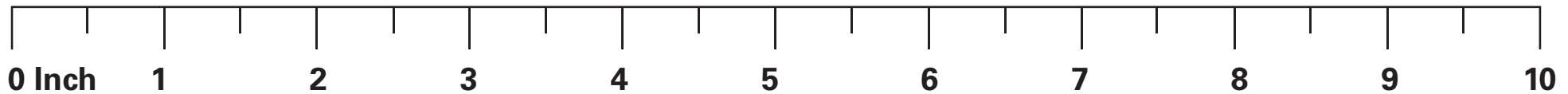

Actual Size: 10 inch (Width) X 5 inch (Height)

Ruler is for scaling purposes. To ensure this print is accurate, match a ruler with the ruler above.