## Power Station II

### Battery Storing information

English • Deutsch • Français • Italiano • Español • Português • Nederlands • Svenska • Dansk • Norsk • Suomi • Ελληνικά • 简体中文 • 繁體中文

English

#### **Caution**

Fully recharge the battery before storage and every three months. Read the complete Storing section within this Addendum before storage.

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**English**  
**Notes:**  

- Not all devices are available in all regions.
- For information on battery run times refer to the Battery/Compatibility List on [www.resmed.com](http://www.resmed.com).

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#### English

The following information replaces the storage information in the Power Station II User Guide.

### Storing

#### CAUTION

Batteries must be charged to 100% indicated by four green LEDs and switched off before storing. You must recharge the batteries to 100% again after three months of storage. When not in use, all Lithium-Ion batteries self-discharge over time. If not periodically recharged (ie, every three months), the RPS II will eventually self-discharge to the point that it can no longer be recharged. If this occurs the RPS II is no longer usable and is unrecoverable.

Store the RPS II in a cool, dry place.

**Note:** *A fully charged battery when left switched on, self-discharges to less than 50% of full capacity within two weeks of storage. Once switched off, the battery self-discharges to less than 50% of full capacity within one month of storage.*

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©2011 ResMed Ltd. 248528/1 2011-07
Designs pending. HumidAire, H4i, H5i, Stellar, VPAP and S9 are trademarks of ResMed Ltd. S9, HumidAire, Stellar and VPAP are registered in U.S. Patent and Trademark Office. VSIII and Elisée are a trademark of ResMed Paris SAS.
**Manufacturer:** ResMed Ltd 1 Elizabeth Macarthur Drive Bella Vista NSW 2153 Australia.
**Distributed by:** ResMed Corp 9001 Spectrum Center Boulevard San Diego CA 92123 USA.