Aclaim[®]2

NASAL MASK

PATIENT INSTRUCTIONS FOR USE

185042991 REV D

Provided for your reference by

cpapXchange.com

Fisher & Paykel
HEALTHCARE

FISHER & PAYKEL HEALTHCARE LTD

INTERNATIONAL	15 Maurice Paykel Place, East Tamaki, Auckland 2013 PO Box 14 348, Panmure, Auckland 1741, New Zealand Tel: +64 9 574 0100 Fax: +64 9 574 0158 Email: info@fphcare.com Website: www.fphcare.com
AUSTRALIA	Tel: +61 3 9879 5022 Fax: +61 3 9879 5232
AUSTRIA	Tel: 0800 29 31 23 Fax: 0800 29 31 22
BENELUX	Tel: +31 40 216 3555 Fax: +31 40 216 3554
CHINA	Tel: +86 (20) 3205 3486 Fax: +86 (20) 3205 2132
FRANCE	Tel: +33 1 6446 5201 Fax: +33 1 6446 5221
GERMANY	Tel: +49 7182 93777 0 Fax: +49 7182 93777 99
INDIA	Tel: +91 80 4123 6041 Fax: +91 80 4123 6044
IRISH REPUBLIC	Tel: 1800 409 011
ITALY	Tel: +39 06 7839 2939 Fax: +39 06 7814 7709
JAPAN	Tel: +81 3 3661 7205 Fax: +81 3 3661 7206
MIDDLE EAST	Tel: +9 71 4 360 0849 Fax: +9 71 4 368 8110
SPAIN	Tel: +34 902 013 346 Fax: +34 902 013 379
SWEDEN	Tel: +46 8 564 76 680 Fax: +46 8 36 63 10
SWITZERLAND	Tel: 0800 83 47 63 Fax: 0800 83 47 54
UK	(EU Authorised Representative) Fisher & Paykel Healthcare Ltd Unit 16, Cordwallis Park, Clivemont Road, Maidenhead, Berkshire SL6 7BU, United Kingdom Tel: +44 1628 626 136 Fax: +44 1628 626 146
Northern Ireland	Tel: +0800 132 189
USA	Tel: +1800 446 3908 or +1 949 453 4000 Fax: +1 949 453 4001

CE 0123

Acclaim^{®2} NASAL MASK PATIENT INSTRUCTIONS FOR USE

THE ACCLAIM^{®2} NASAL MASK IS INTENDED FOR MULTIPLE PATIENT OR SINGLE PATIENT ADULT USE BY INDIVIDUALS WHO HAVE BEEN DIAGNOSED BY A PHYSICIAN AS REQUIRING CPAP OR BI-LEVEL VENTILATOR TREATMENT.

FITTING YOUR MASK

- Remove plastic cover from Silicone Seal (C).
 - Holding your mask in one hand, spread the Headgear (K) with your free hand, just enough to fit comfortably over your head.
 - Lightly hold the mask against your face, covering the nose, then slowly pull the Headgear over your head.
 - Attach the Headgear to the Glider™ Strap (E). Pull the lower straps as low as possible down the back of your neck. Gently tighten straps to remove slack. Connect to the CPAP/Bi-Level machine set at prescribed pressure.
 - Adjust the tension in each strap for comfort and to ensure there are no leaks.
 - If necessary, tighten the lower straps to eliminate any leaks (avoid over-tightening the straps).
 - Use the Silicone Seal that provides the best fit for you. Small and Large are both provided with your mask.
 - When you need to remove your mask, unhook the Headgear from the Glider™ Strap.
- FITTING TIPS**
- Forehead pads should rest gently on the forehead without top straps being pulled too tight.
 - If there are any leaks after gently tightening the top and lower straps, pull the mask slightly out from the face while straps are still attached. This will allow Silicone Seal to inflate with air. Put the mask back onto the face.

CLEANING YOUR MASK

To ensure continuing safety and comfort from your Acclaim^{®2} Nasal Mask:

BEFORE FIRST USE:

- Hand-wash the Headgear in pure soap dissolved in lukewarm water. Do not soak for longer than 10 minutes, then rinse in fresh water and allow to dry out of direct sunlight.

BEFORE EACH USE:

- Inspect the mask for deterioration. Do not use if damaged.

AFTER EACH USE:

- Wash all parts except the Foam Cushion (D), Headgear (K) and Diffuser (G) in pure soap dissolved in lukewarm water. Do not soak for longer than 10 minutes, then rinse in fresh water and allow to dry out of direct sunlight. Wipe the Foam Cushion clean with a damp cloth or sponge and leave to dry.

The Diffuser (G) should not be washed or rinsed. Simply replace it with a new Diffuser if it becomes noisy or deteriorates. It should last two to three months. Small dots may appear on the Diffuser - these do not affect its performance. The Acclaim^{®2} Nasal Mask may be used with or without the Diffuser and Diffuser Cap (H).

EVERY 7 DAYS:

- Hand-wash the Headgear in pure soap dissolved in lukewarm water. Do not soak for longer than 10 minutes, then rinse in fresh water and allow to dry out of direct sunlight.

CAUTIONS

- Do not soak the mask for longer than 10 minutes.
- Do not wash the mask in the dishwasher.
- Do not clean the mask with products containing alcohol, anti-bacterial agents, antiseptic, bleach, chlorine or moisturizer.
- Do not store the mask in direct sunlight.

The above actions may deteriorate or damage the mask and shorten its life. If your Acclaim^{®2} Nasal Mask weakens or cracks, discontinue use and seek replacement immediately.

ASSEMBLING YOUR MASK

Your Acclaim^{®2} Nasal Mask is fully assembled and ready to use. However, when you need to clean your mask, use the following instructions for reassembly. Refer to the diagram on the left.

- Place the Foam Cushion (D) into the Mask Base (A).
- Starting at the top of the mask, fit the Silicone Seal (C) over the Foam Cushion and onto the Mask Base, finishing at the bottom.
- Fit the small end of the Flexitube Assembly (B) onto the Mask Base.
- Fit the Diffuser (G) into the Diffuser Cap (H) and snap the assembly onto the Mask Base.
- Lay the Headgear (K) flat, with the loop side down. Position the assembled mask over the Headgear and attach the five straps to the corresponding slots in the Forehead Rest (I) and Glider™ Strap (E). Finally, secure the Flexitube Assembly with the tab attached to the top strap.

OXYGEN/PRESSURE PORT CONNECTOR

If pressure readings and/or additional oxygen are required, an Oxygen/Pressure Port Connector is available (Code 900HC452).

- Connect the large end of the Connector (T) to the Swivel (I) on the mask and the small end of the Connector to the main Breathing Tube (R).
- Open the Port Cap(s) (P) and place the pressure and/or oxygen tube (S) firmly over the Port(s) (Q).

(The two ports are identical; either port may be used for oxygen or pressure measurement. Only remove the Port Cap from ports that are being used.)

Note: At a fixed flow rate of supplemental oxygen flow, the inhaled oxygen concentration will vary, depending on the pressure settings, patient's breathing pattern, mask selection, and the leak rate.

Warning: If oxygen is used with CPAP/Bi-Level machine, the oxygen flow must be turned off when the CPAP/Bi-Level machine is not operating.

Explanation of Warning: When the CPAP/Bi-Level device is not in operation, and the oxygen flow is left on, oxygen delivered into the ventilator tubing may accumulate within the CPAP/Bi-Level machine enclosure. Oxygen accumulated in the CPAP/Bi-Level machine enclosure will create a risk of fire.

Warning: Do not smoke while using or in the vicinity of someone using oxygen with this mask.

Warning: Do not use Polyvinyl Chloride (PVC) pressure and/or oxygen tubing. Use silicone tubing instead.

Explanation of Warning: Contact with PVC can cause premature cracking of ports.

WARNINGS

- This mask should be used only with CPAP or Bi-Level equipment recommended by your physician or respiratory therapist. A mask should not be used unless the CPAP or Bi-Level machine is turned on and operating properly.
- Explanation of Warning:** CPAP and Bi-Level machines are intended to be used with special masks which have exhaust flow holes to allow continuous flow of air out of the mask. When the CPAP/Bi-Level machine is turned on and functioning properly, new air from the CPAP/Bi-Level machine flushes the exhaled air out through the exhaust flow holes. However, when the CPAP/Bi-Level machine is not operating, enough fresh air will not be provided through the mask, and exhaled air may be re-breathed. Re-breathing of exhaled air for longer than several minutes can in some circumstances lead to suffocation.
- At low CPAP/Bi-Level pressures the flow through the exhalation ports may be inadequate to clear all exhaled gas from the tubing. Some re-breathing may occur.
- Do not block the gas exhalation ports or use alternative Diffusers, other than those manufactured by Fisher & Paykel Healthcare. Before use, always ensure that gas is flowing out the exhalation ports or through the Diffuser (if it is being used).
- Do not use the mask if vomiting or feeling nauseous.
- Discontinue using the mask if there is an allergic reaction to any part of the mask. Consult your physician if this occurs.
- If reusing the Acclaim^{®2} Nasal Mask between different patients, refer to the Multi-Patient Cleaning Instructions for the Acclaim^{®2} Nasal Mask.

CAUTIONS

- Use the mask for its intended use only as directed in this booklet.
- US Federal Law restricts this device to sale by, or on order of, a physician.

The Acclaim^{®2} Nasal Mask and accessories are latex free. Many people find CPAP/Bi-Level therapy causes dryness of the nose and throat. Heated humidification can substantially reduce these symptoms.

US PATENT NUMBER 6 615 834
OTHER PATENTS PENDING

FOR ADDITIONAL INFORMATION CONTACT YOUR LOCAL HOMECARE PROVIDER

TECHNICAL SPECIFICATIONS

EXHAUST FLOW

Nasal masks for CPAP and Bi-Level machines have exhaust flow holes to expel the air that you exhale from the mask. The Acclaim^{®2} Nasal Mask has a unique Diffuser system that is designed to diffuse the air that is expelled from the mask. This reduces the draft.

The Diffuser does not significantly change the amount of air flowing from the mask, therefore the mask can be used with or without the Diffuser.

PRESSURE (cmH ₂ O)	3	5	7	9	11	13	15	17	19	21	23	25
Flow (L/MIN)	15	19	23	26	28	31	33	35	37	39	41	43

RESISTANCE TO FLOW

Pressure drop through mask @ 50 l/min 0.1 cmH₂O
Pressure drop through mask @ 100 l/min 0.5 cmH₂O

MASK DEAD SPACE
80cc

OPERATING CONDITIONS

- Operating pressure is 3-25 cm H₂O
- Operating temperature range is 5-50°C (40-120°F)

MULTI-PATIENT CLEANING INSTRUCTIONS

Your Acclaim^{®2} Nasal Mask is provided patient-ready. To reuse between patients please follow the guidelines below.

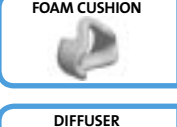
CLEANING / DESOLING

- Before disinfection, remove visible soil by disassembling and thoroughly cleaning the mask with a small brush under running potable tap water.

FAILURE TO PROPERLY CLEAN THIS DEVICE MAY RESULT IN INADEQUATE DISINFECTION.

HIGH-LEVEL DISINFECTION PARAMETERS

- NB: For autoclave cycles, components should be enclosed in a sterilization peel pouch.

MASK BASE 	AUTOCCLAVING Autoclave, 30 minutes @ 121°C (250°F). OR THERMAL DISINFECTION Soak parts in hot-water bath for 30 minutes at 70°C (158°F). OR STERAD[®] 100S Follow manufacturer's instructions for Sterad [®] 100S sterilization system. OR CIDEX[®] OPA Follow manufacturer's instructions for high-level disinfection. Immerse for 30 minutes at 20°C (68°F). Rinse thoroughly in potable water. OR SPOROX[®] II Follow manufacturer's instructions for high-level disinfection. Immerse for 6 hours at 20°C (68°F). Rinse thoroughly in potable water.
FLEXITUBE & SWIVEL 	THERMAL DISINFECTION Soak parts in hot-water bath for 30 minutes at 70°C (158°F). OR STERAD[®] 100S Follow manufacturer's instructions for Sterad [®] 100S sterilization system. OR CIDEX[®] OPA Follow manufacturer's instructions for high-level disinfection. Immerse for 30 minutes at 20°C (68°F). Rinse thoroughly in potable water. OR SPOROX[®] II Follow manufacturer's instructions for high-level disinfection. Immerse for 6 hours at 20°C (68°F). Rinse thoroughly in potable water.
HEADGEAR 	Hand-wash the Headgear in pure soap dissolved in lukewarm water. Dry out of direct sunlight.
FOAM CUSHION 	Replace with new part.
DIFFUSER DIFFUSER COVER 	If continuing use with the Diffuser, replace with a new part.*

*Use of the Diffuser and Diffuser Cap is optional on the Acclaim^{®2} Nasal Mask. Disclaimers: It is the user's responsibility to qualify any deviations from the recommended method of reprocessing. The Acclaim^{®2} Nasal Mask can be reprocessed 20 times using the above methods. Do not use if damaged.

CONTACT YOUR LOCAL HOMECARE PROVIDER FOR ADDITIONAL INFORMATION.

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