

HANS RUDOLPH inc.

MASK SIZING GAUGE

CHECK ADOBE PRINT SETTINGS BEFORE PRINTING

PAGE SCALING: NONE

IF PREVIEW IS OUTSIDE PRINTING BORDER

SELECT AUTO-ROTATE & CENTER

The diagram illustrates the mask sizing gauge with height measurements for different mask sizes. The measurements are as follows:

Mask Size	Height Measurement (cm)
PETITE (P)	3.60
EXTRA-SMALL (XS)	4.00
SMALL (S)	4.40
MEDIUM (M)	4.90
LARGE (L)	5.40

Additional measurements shown in the diagram include 5.90 and 4.90.

The illustration shows a person's head in profile with the mask sizing gauge applied to the face. The gauge is a vertical strip with horizontal bands. The top of the gauge is aligned with the nasal root depression (sellion), and the bottom is aligned with the bottom of the chin. The gauge is used to determine the mask size that will best fit the patient.

MASK SIZING GAUGE

Directions: Length measurements are used to determine the size of the Hans Rudolph Oro-Nasal mask that will best fit the patient. For measurement: the patient's facial muscles should be relaxed and jaw closed. Place the Gauge tip to the deepest point of the nasal root depression (sellion) and note the size indication where the bottom of the chin exits the gauge. The gray band widths indicate the size ranges.

This gauge is a reference to aid in sizing and not an exact measurement as other factors may need to be taken into account.

IF BETWEEN SIZES, WE RECOMMEND YOU USE THE SMALLER SIZE.

HANS RUDOLPH, inc.

PETITE (P)
EXTRA-SMALL (XS)
SMALL (S)
MEDIUM (M)
LARGE (L)

cpapXchange.com
Provided for your reference by

www.rudolphkc.com