


Mirage FX components

Mirage™ FX NASAL MASK

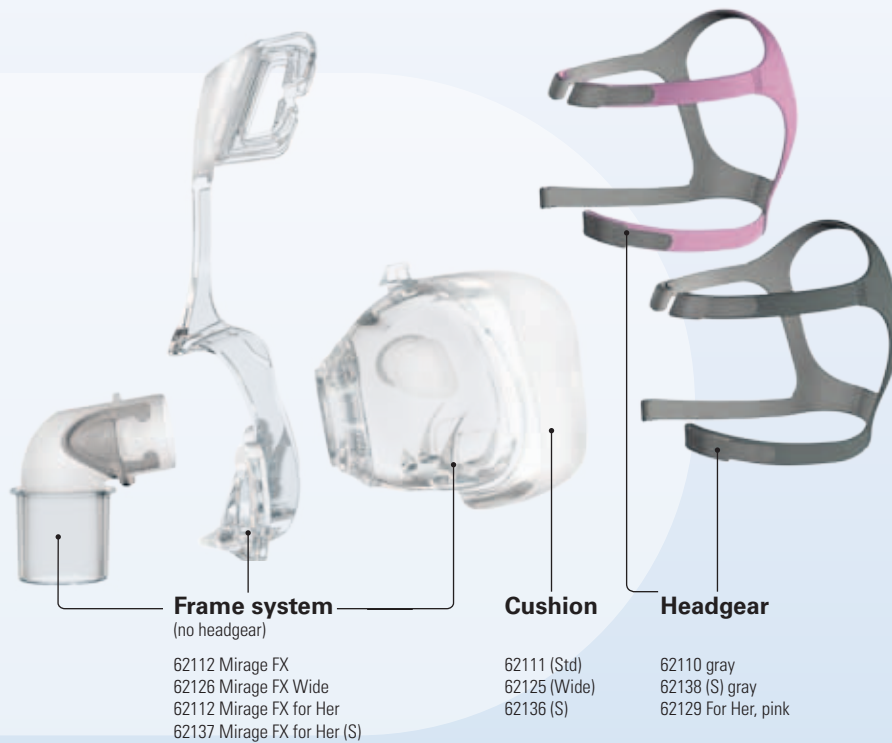

PRODUCT CODES

Mirage FX Nasal Mask

Complete System

Size	Part Number
Mirage FX	62103
Mirage FX Wide	62118
Mirage FX for Her	62128
Mirage FX for Her (S)	62109

Reimbursement Codes

HCPCS	Code Descriptor
A7034	Nasal mask used with positive airway pressure device, 1 per 3 months
A7032	Replacement cushion for nasal mask, 2 per month
A7035	Headgear used with positive airway pressure device, 1 per 6 months

⚠ For details, refer to the **Mirage FX Nasal Mask User Guide**.

Fitting



- Prior to fitting the mask, remove the elbow from the mask by pressing the squeeze-tabs.



- Connect the elbow to the device air tubing.



- With both lower headgear straps released, hold the mask against the face and pull the headgear over the head.



- Bringing the lower straps below the ears, loop the headgear into the lower hooks on the mask frame.



- Unfasten the straps, pull the upper headgear straps evenly until they are comfortable and firm, and reattach the straps.
- Repeat with lower straps.



- Attach the combined elbow assembly and air tubing to the mask by pressing the squeeze-tabs and pushing into the mask frame.
- Turn on the airflow device.