

Fitting

Swift FX Bella



- 1
- Hold the pillows at your nose, ensuring the ResMed logo is above the pillows facing up. Pull one strap around and behind your ear.
 - Ensure the strap lies flat and is not twisted.
 - Repeat with the other strap.



- 2
- Adjust the mask fit by pulling the upper straps along the notches until a comfortable fit is achieved. Repeat with the lower straps. Do not overtighten.



- 3
- Connect the free end of the short tube assembly to the device air tubing.

Swift FX for Her



- 1
- Holding the pillows at your nose, pull the headgear over your head.
 - Headgear should sit midway between the ears and eyes.



- 2
- Adjust the top strap first.
 - Pull the strap through the buckle one notch at a time.
 - Do not overtighten.



- 3
- Adjust the backstrap to fine-tune the fitting if necessary.
 - Do not overtighten.
 - The backstrap can be positioned over or under the hair.



- 4
- Swift FX pillows are designed to provide a natural fit.
 - Minor adjustment can help position the pillows at the most comfortable angle in the nose.
 - Connect the free end of the short tube assembly to the device air tubing.

Optional soft wraps

- Wrap each of the soft wraps around either side of the headgear, folding the top fastener over the lower one.
- The soft wraps should be positioned on the cheeks, with the fasteners on the outside.



S9™ Series
 |More. Comfort.



Combine a Swift FX for Her mask with ResMed's S9 AutoSet™ for Her therapy device for a comfortable, quiet, lifestyle-friendly system.

Swift™ FX Bella
 NASAL PILLOWS SYSTEM

Less mask. More you.