

## Fitting Guide



- Ensure bottom of cushion opening aligns with the bottom of your nose then place the mask over your nose.
- Then pull headgear over your head so it sits midway between ears and eyes.



- Adjust the top strap first by pulling the strap through the buckle, one notch at a time, until the mask is comfortably over your nose.
- Do not overtighten.



- The back strap should sit just above the ears at the back of the head.
- Adjust both fasteners on the fabric back strap at the same time.
- Ensure back strap sits over your hair.



- Ensure mask is comfortably sealed.



- Adjust the nasal cushion until it sits comfortably on your nose.
- Connect the free end of the short tube assembly to the air tubing of your device, then turn it on.



- Check for leak and adjust if needed. If any air is leaking out then readjust the back strap before readjusting the top strap.
- Your mask is now ready to use.

## Optional Soft Wraps



- Wrap each of the soft wraps around either side of the headgear, folding the top fastener over the lower one.
- The soft wraps should be positioned on your cheeks, with the fasteners on the outside.

*Note: For additional information, refer to the Swift FX Nano / Swift FX Nano for Her Nasal Mask User Guide.*

## Swift™ FX Nano NASAL MASK

## Fitting Guide



- Ensure bottom of cushion opening aligns with the bottom of your nose then place the mask over your nose.
- Then pull headgear over your head so it sits midway between ears and eyes.



- Adjust the top strap first by pulling the strap through the buckle, one notch at a time, until the mask is comfortably over your nose.
- Do not overtighten.



- The back strap should sit just above the ears at the back of the head.
- Adjust both fasteners on the fabric back strap at the same time.
- Ensure mask is comfortably sealed.



- Position the back strap over or under your hair to suit your hairstyle preference.



- Adjust the nasal cushion until it sits comfortably on your nose.
- Connect the free end of the short tube assembly to the air tubing of your device, then turn it on.



- Check for leak and adjust if needed. If any air is leaking out then readjust the back strap before readjusting the top strap.
- Your mask is now ready to use.

## Optional Soft Wraps



- Wrap each of the soft wraps around either side of the headgear, folding the top fastener over the lower one.
- The soft wraps should be positioned on your cheeks, with the fasteners on the outside.

*Note: For additional information, refer to the Swift FX Nano / Swift FX Nano for Her Nasal Mask User Guide.*

**Swift™ FX Nano for Her**  
 NASAL MASK



**Actual Size: 5½" (Width)  
x 7" (Height)**


Ruler is for scaling purposes.

**How to use the fitting template:**

1. Cut out the template along the dotted line
2. Follow directions on template to determine appropriate cushion size

**Swift™ FX Nano**  
NASAL MASK

**Swift™ FX Nano for Her**  
NASAL MASK

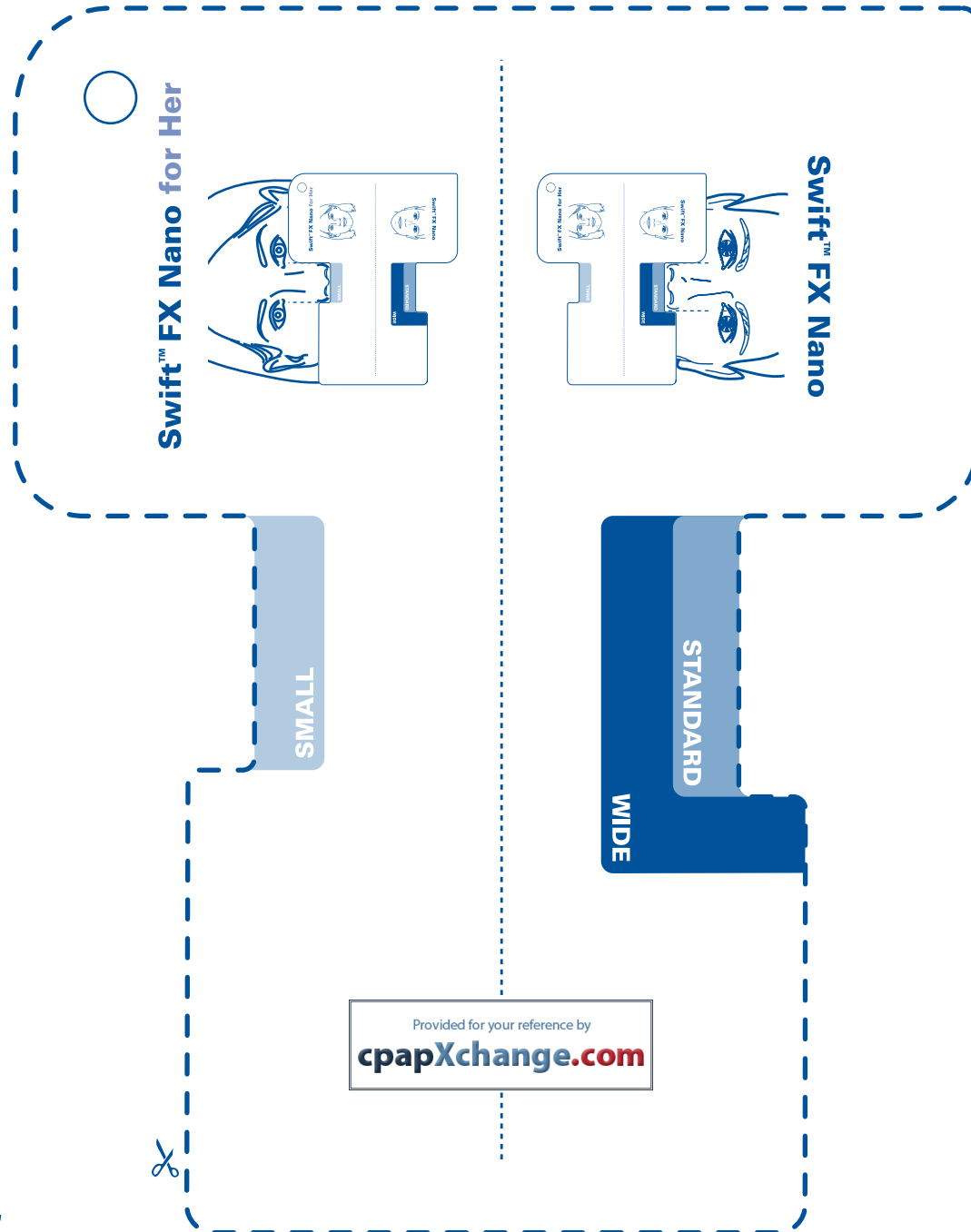

CE 0123 Rx Only