

### Intended Use

The travel kit accessory is a reduced size carry case with a 65W power supply. The travel kit accessory is intended to support operation of the DreamStation CPAP and BiPAP sleep therapy devices without the optional humidifier. The 65W power supply includes a USB port intended to charge extra items, including smartphones or other electronics.

### Package Contents

The travel kit carry case is designed to fit a therapy device, 65W power supply, mask and hose, but not a humidifier. Note that the humidifier and heated tube will not operate when using the 65W power supply, and an icon will appear on the screen.

- Carry Case
- Shoulder Strap
- 65W Power Supply



### Warnings and Cautions

**Important!** Refer to the therapy device manual for additional warnings and cautions.



#### Warnings




- The operator should read and understand the instructions for use before using the 65W power supply.
- The travel kit accessory has no serviceable parts.
- Use only power cords supplied by Philips Respironics for the 65W power supply. Use of power cords not supplied by Philips Respironics may cause overheating or damage to the 65W power supply and may result in increased emissions or decreased immunity of the equipment or system.
- Periodically inspect electrical cords and cables for damage or signs of wear. Discontinue use and replace if damaged.
- Be sure to route the power cord to the outlet in a way that will prevent the cord from being tripped over or interfered with by chairs or other furniture.
- The 65W power supply is activated when the power cord is connected.

#### Cautions

- Do not use extension cords with the 65W power supply.
- Do not place the 65W power supply in or on any container that can collect or hold water.

### Symbols

Symbol	Definition
	Consult accompanying instructions
<b>IP22</b>	Drip proof equipment
	For Indoor Use Only.

Symbol	Definition
	Class II (Double Insulated)
	Caution, consult accompanying documents.
	Separate collection for electrical and electronic equipment per EC Directive 2012/19/EU

## Supplying AC Power to the Device

Refer to the DreamStation therapy device manual.

## Temperature and Storage Information

	Operation	Transport & Storage
<b>Temperature:</b>	41° to 95° F (5° to 35° C)	-4° to 140° F (-20° to 60° C)
<b>Humidity:</b>	15% to 95% non-condensing	15% to 95% non-condensing
<b>Atmospheric Pressure:</b>	101 to 77 kPa (0 - 2286 m / 0 - 7500 ft)	101 to 77 kPa (0 - 2286 m / 0 - 7500 ft)

## Disposal

Dispose of this device in accordance with local regulations.

## Product Support

Should you experience trouble with this equipment or require assistance setting up, using, or maintaining the equipment, contact your home care provider or Philips Respironics at +1 724-387-4000 or +49 815293060.

## Specifications

### Service Life

The expected service life of the 65W power supply is 5 years.

### IEC 60601-1 Classification

Type of Protection Against Electric Shock: Class II Equipment  
Degree of Protection Against Electric Shock: Type BF Applied Part  
Degree of Protection against Ingress of Water: Drip Proof, IP22  
Mode of Operation: Continuous

### Electrical

AC Power Consumption: 100 – 240 VAC, 50/60 Hz, 2.0-1.0 A

**Note:** 65W power supply is part of the medical electrical equipment.

DC Power Output: 12 VDC, 4.58 A

USB Port Output: 5.0V, 2.0 A

Fuses: There are no user-replaceable fuses.