

SLEEP COMFORT CARE PAD

Cleaning and Application Guide

Provided for your reference by
cpapXchange.com

S
Comfort Care
**SLEEP
COMFORT
CARE
PAD**
DISTRIBUTED BY SEQUAL TECHNOLOGIES INC.

1
STEP



Clean pad with hand soap before each use.

2
STEP

Rinse pad with warm to hot tap water. The water should NOT burn your hands.



3
STEP



Blot pad dry using a towel.

4
STEP

Apply pad to your face. Place tip of pad at eyebrow level, and wings of pad 1/4" below bottom of eyes.



5
STEP



Apply your CPAP mask.

6
STEP

Ensure a secure fit.

